ACUMED®

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ACUMED FINGER TRAPS AND ARM STRAPS: The Arc Wrist Tower finger traps and arm straps are disposable, single use instruments only available for use with the Acumed Wrist Tower. These instruments should be discarded after each use.

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occur immediately after completion of the surgical procedure. Excess blood or debris should be wiped off to prevent it from drying onto the surface. Using an enzymatic-cleaning product such as Enzol, prepare the cleaning solution according to manufacturer's instructions.

NOTE: Anodized aluminium must not come in contact with certain cleaning or disinfectant solutions. Avoid strong alkaline cleaners and disinfectants or solutions containing iodine, chlorine or certain metal salts. Also, in solutions with pH values above 11, the anodisation layer may dissolve.

Avoiding prolonged soaking, submerge the device in the enzymatic cleaning solution for 5 minutes. After 5 minutes, use a soft bristle brush to remove any detergent residue. Instruments with lumens also should be held under water (larger side up) ensuring the lumen is flushed with water. Place the device in an ultrasonic cleaner filled with deionized water, ensuring that there is no air trapped in any of the crevices by gently agitating the device. Sonicate the device for 5 minutes. Remove the device from the ultrasonic cleaner and rinse in water by submerging it completely. Gently wipe the device with a clean, dry cloth, then air dry.

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Sterilization Methods

Gravity Displacement Autoclave	Full Cycle Time: 270° F (132° C) for 30 minutes Dry Time: 60 minutes
Pre-vacuum Autoclave	Full Cycle Time: 270° F (132° C) for 4 minutes Dry Time: 60 minutes

- Consult your equipment manufacturer's written instructions for specific sterilizer and load configuration instructions.
- Follow current AORN® Recommended Practices for Sterilization in Perioperative Practice Settings® and ANSI/AAMI ST79: 2006 – Comprehensive guide to steam sterilization and sterility assurance in health care facilities.
- Flash sterilization is not recommended, but if used, should only be performed according to requirements of ANSI/AAMI ST79: 2006 – Comprehensive guide to steam sterilization and sterility assurance in health care facilities.

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ARC WRIST TOWER INSTRUMENT AND ACCESSORY INFORMATION

FOR THE PERSONAL ATTENTION OF THE OPERATING SURGEON

DESCRIPTION: Proper care and inspection after each use of the ACUMED ARC Wrist Tower and accessories is critical.

INFORMATION FOR USE:

ACUMED ARC WRIST TOWER: The Arc Wrist Tower is reusable; however it has a limited life span. Prior to and after each use, it must be inspected where applicable for wear, damage, proper cleaning, corrosion and integrity of the connecting mechanisms.

ACUMED FINGER TRAPS AND ARM STRAPS:

The Arc Wrist Tower finger traps and arm straps are disposable, single use instruments only available for use with the Acumed Wrist Tower. These instruments should be discarded after each use.

WARNINGS: For safe effective use of any Acumed instrument, the surgeon must be familiar with the instrument, the method of application, and the recommended surgical technique. Instrument breakage or damage can occur when an instrument is subjected to excessive loads, or speeds. Particular care should be taken not to apply excessive torque to components during assembly or disassembly. In addition, all portions of the traction tower must be cool to the touch prior to placing the patient's arm in it. The patient must be cautioned, preferably in writing as to the risks associated with these types of instruments.

PRECAUTIONS: A Single Use Surgical Instrument shall never be reused. Previous stresses may have created imperfections, which can lead to a device failure. Protect instruments against scratching and nicking, such stress concentrations can lead to failure.

CLEANING: Acumed Instruments and Accessories must be thoroughly cleaned before reuse. Decontamination of reusable instruments or accessories should

occur immediately after completion of the surgical procedure. Excess blood or debris should be wiped off to prevent it from drying on to the surface. Using an enzymatic-cleaning product such as Enzol, prepare the cleaning solution according to manufacturer's instructions.

NOTE: Anodized aluminum must not come in contact with certain cleaning or disinfectant solutions. Avoid strong alkaline cleaners and disinfectants or solutions containing iodine, chlorine or certain metal salts. Also, in solutions with pH values above 11, the anodization layer may dissolve.

Avoiding prolonged soaking, submerge the device in the enzymatic cleaning solution for 5 minutes. After 5 minutes, use a soft bristle brush to remove any detergent residue. Instruments with lumens also should be held under water (larger side up), ensuring the lumen is flushed with water. Place the device in an ultrasonic cleaner filled with deionized water, ensuring that there is no air trapped in any of the crevices by gently agitating the device. Sonicate the device for 5 minutes. Remove the device from the ultrasonic cleaner and rinse in water by submerging it completely. Gently wipe the device with a clean, dry cloth, then air dry.

NOTE: Even surgical instruments manufactured from high-grade stainless steel must be dried thoroughly to prevent rust formation, and all devices must be inspected for cleanliness of surfaces, joints, and lumens, proper function, and wear and tear prior to sterilization.

STERILITY: The Acumed Arc Wrist Tower and accompanying Acumed finger traps and arm straps are provided non-sterile. Sterilization may be performed for these items collectively or individually when adhering to the below sterilization methods. To sterilize the finger traps and arm straps, remove them from their plastic packaging and place them in either of the Arc Wrist Tower trays.

Sterilization Methods

Gravity Displacement Autoclave	Full Cycle Time: 270° F (132° C) for 30 minutes Dry Time: 60 minutes
Pre-Vacuum Autoclave	Full Cycle Time: 270° F (132° C) for 4 minutes Dry Time: 60 minutes

- Consult your equipment manufacturer's written instructions for specific sterilizer and load configuration instructions.
- Follow current AORN "Recommended Practices for Sterilization in Perioperative Practice Settings" and ANSI/AAMI ST79: 2006 – Comprehensive guide to steam sterilization and sterility assurance in health care facilities.
- Flash sterilization is not recommended, but if used, should only be performed according to requirements of ANSI/AAMI ST79: 2006 – Comprehensive guide to steam sterilization and sterility assurance in health care facilities.

STORAGE INSTRUCTIONS: Store in a cool dry place and keep away from direct sunlight. Prior to use, inspect product package for signs of tampering, or water contamination.

CAUTION: For Professional Use Only

CAUTION: Federal Law (USA) restricts this product sale by or on the order of a physician or hospital.